

One Disc!  
One-derful Polish!  
**Polish & Go!**



**PoGo**  
New™

One Step Diamond Micro-Polisher

PoGo™ One Step  
Diamond  
Micro-Polishers  
are changing the  
way we polish  
composites.

PoGo, the one step  
diamond micro-polisher, is  
designed to create incre-  
dible luster on even the  
hardest advanced hybrid  
composite. While other  
polishing systems require  
multiple discs, steps and  
several minutes<sup>1</sup>, PoGo  
diamond micro-  
polishers allow  
you to “polish  
and go” in  
one step, in  
a fraction of  
the time.

With PoGo diamond  
micro-polishers, the final  
result is exceptional luster  
and more time to admire  
your work.

1. (Survey - July 2001) Dentists spend an average of 6.5 minutes polishing a direct composite lateral veneer, and an average of 6 steps.

Forget 3 or more steps...

...Remember One!

# PoGo™

One Step Diamond Micro-Polisher




Change the way you polish - you'll love it!

- One disc
- Less time
- Exceptional luster
- Polishes all composites
- Polish and go!

### Roughness Comparison


Effect of Final Step on Surface (by Material)

PoGo diamond micro-polishers give you a higher luster on different restorative materials.<sup>2</sup>

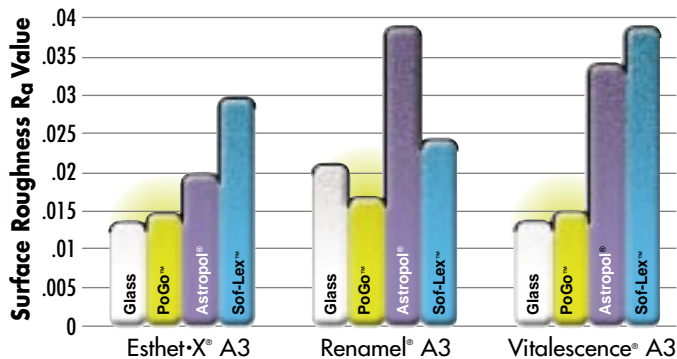

"Glass" refers to the smoothness of composite cured against a piece of glass. The graph shows the smoothness after utilizing the last component of each polishing system.

### Comparison of Sof-Lex™ Superfine polishing disc to PoGo diamond micro-polisher on human enamel for 30 seconds at 1000x.

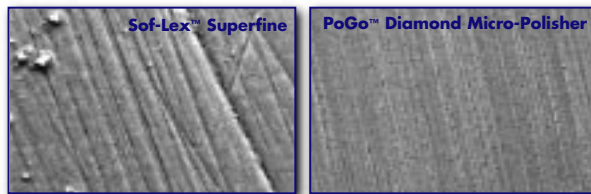

### Easy, Fast Technique



Start with a surface on your restoration that is finished to form, smooth and defect-free. Use the broad, flat side of the PoGo™ diamond micro-polisher and start polishing with light pressure. Sweep PoGo diamond micro-polisher over the surface of the restoration and luster will begin to appear. Decrease pressure to a "feather-like" touch to bring out the highest luster.



Photography by Dr. Jeff Blank

### Ordering Information

**PoGo Refill Package** 662010Y  
30 Diamond Polishing Discs

**PoGo CD-ROM** K 791.00.503  
Technique Video



For more information please contact:

**DENTSPLY DeTrey GmbH**  
De-Trey-Str. 1  
D-78467 Konstanz  
Tel. (07531) 5830  
www.dentsply.de

**DENTSPLY**  
DeTrey

<sup>2</sup> Data on file.

Astropol, Sof-Lex, Renamel and Vitalescence are not registered trademarks of DENTSPLY International.